


Dryers/Accessories

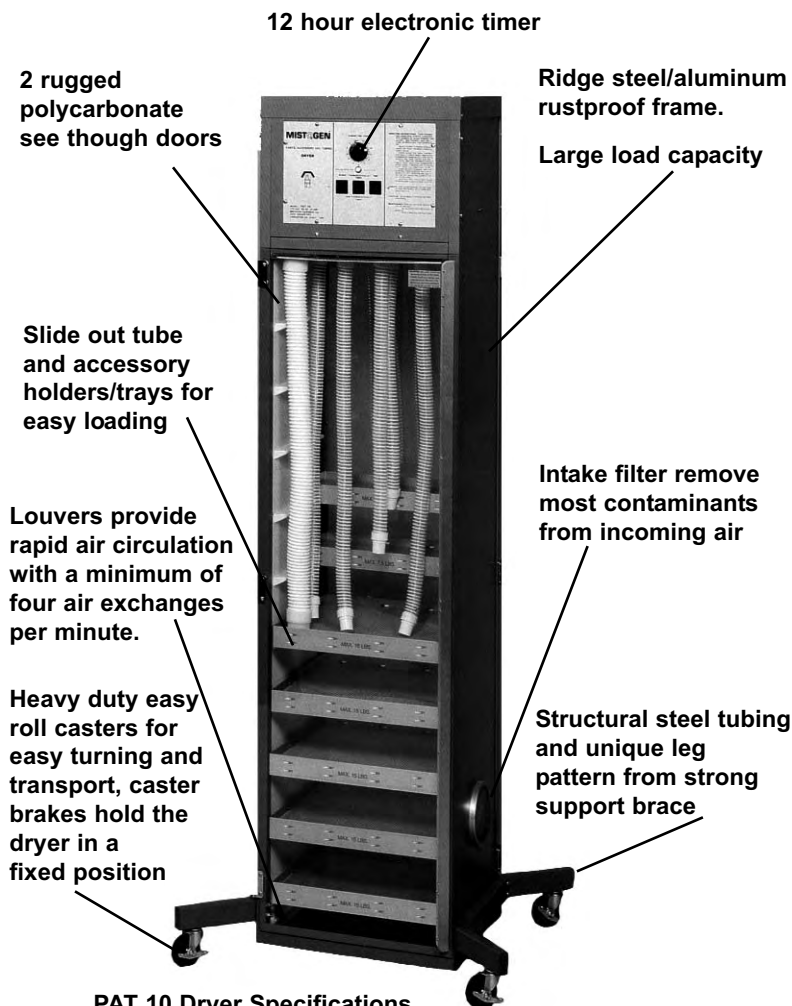

PAT 10 Dryer Versatile

The PAT 10 operates as a dryer for hoses, parts and accessories. In fact, the PAT 10 is designed with whole and half trays, plus a variety of tube holders. The PAT 10 enables you to dry tubes and parts at the same time! Permanent and mobile units both available.

Durable

Mistogen's PAT 10 Dryer has a rigid steel/aluminum frame that is rust-proof and scratch-resistant. The two see-through doors on the PAT 10 are manufactured of rugged polycarbonate. Three hinges and latches per door form a tight seal for efficient operation. The PAT 10's unique leg pattern, which is manufactured from structural steel tubing, forms a strong support base.

A Choice of Two Drying Modes

The PAT 10 Dryer provides you with the choice of two drying modes - with a simple flip of a switch. The first mode is thermostatic timed. The second energy saving mode is nonthermostatic timed.

In the thermostatic timed mode, the PAT 10 timer will only run as long as the inside cabinet temperature is above 115°F (46° C). If the temperature is below 115° F (46° C). the timer will stop and not start again until the temperature reaches 115° F (46° C). This thermostatic timed mode is particularly useful for drying items that tend to trap moisture and thus may require a constant temperature for drying expediency.

In the energy saving timed mode, as soon as you turn on the PAT 10 and set the timer, it starts running. This mode is useful for products that do not require intense drying and reduces power consumption.

For safety during both modes, there is a thermostat working to ensure that the temperature does not go above 150° F (66° C).

PAT 10 Dryer Specifications Cabinet

Width:	20" (50.8 cm)
Depth:	20" (50.8 cm)
Height:	80-5/8" (204.8 cm)
Weight:	164 lbs. (74.5 kg)
Energy Requirements:	
Power:	115 VAC, 60 Hz, 15A Maximum
Power Dissipation:	1380W (4710 Btu/hr)
Equipped with hospital grade plug and circuit breaker	
Drying Compartment:	
Width:	18" (45.7 cm)
Depth:	18" (45.7 cm)
Height:	64" (162.6 cm)
Tray Size	
Full	16½" x 16½" (42 cm x 42 cm)
Half	16½" x 8-3/8" (42 cm x 21.3 cm)
Tube Holders	
Standard	#6714 holds (12) 2-1/8 tubes (5.4 cm) (54) and 1-5/16" (3.3 cm) tubes

Ordering Information

PAT 10

Part #	Description
AW-6700	PAT 10 Cabinet Dryer, castered, 5 full trays and 10 Half Trays with Electric Timer and automatic Heat Control.
AW-6745	Tube holder for (12) 2-1/4" tubes and (54) 1-5/16" tubes. 115V, 60 Hz
AW-3820	PAT 10S Cabinet Dryer stationary, same as above less casters.
AW-3821	Aerosol Delivery Tube, 24" x 1¼"
AW-3822	Aerosol Delivery Tube, 36" x 1¼"
AW-3823	Aerosol Delivery Tube, 48" x 1¼"
AW-4185	Aerosol Delivery Tube, 30" x 1¼"
AW-4188	Disposable Aerosol Tubing 80" x ¾"
AW-4188	¾" Disposable Tubing, 100'

Dryers/Accessories

TAD 25 Dryer Compact/Versatile



Mistogen's TAD 25 is a quiet dryer that efficiently dries up to 25 tubes up to 72" long, in less than 20 minutes. Plus, with the TAD 25 carrying trays, bracket and open end curtain, the TAD 25 converts easily to a small parts dryer. Tubes and accessories can be dried simultaneously in less than 20 minutes.

Because the TAD 25 is compact and wall mounted, it uses very little department space. The TAD 25 is ideal for any size hospital, from 20 to 1,000 beds. Besides Respiratory Care, the TAD 25 may be used in Central Supply or Anesthesia Departments and for Home care.


Well Designed Operation

The TAD 25 is easy to operate! Two rocker switches on the front panel control AIR and HEAT. Both switches light up red to show the unit is on. The AIR switch is a master, it must be on to get heat and prevents overheating. The TAD 25 can operate with or without heat. A heat adjustment knob allows you flexibility in the amount of heat you use. On the back of the TAD 25 are four black rubber feet that stabilize the TAD 25 when mounted against the wall.

Width: 10" (25.4 cm)
 Depth: 10" (25.4 cm)
 Height: 13" (33.0 cm)
 Weight: 17 lbs. (7.7 kg.)
 Energy: 115 VAC, 60 Hz (230 V, 50 Hz)
 Requirements: 7A 3.5A
 Power: 805W (2747 Btu/hr). Equipped with
 Dissipation: hospital grade plug and circuit breaker.

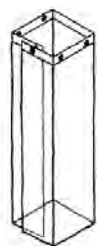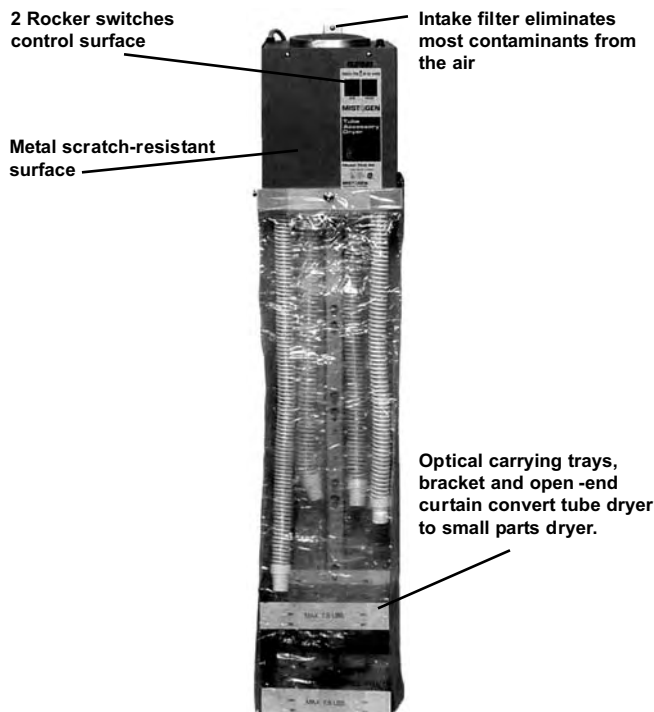

AW- 1803
Removeable Air
Curtain Assembly

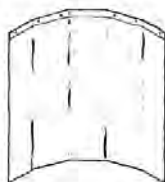

AW- 1835
Air Flow Curtain



AW- 1805 Bracket Strip
Assembly (per section)



AW- 1875 Holding Tray
Assembly 10" x 10"
(25.4 cm x 25.4 cm)



AW- 1844 6" Filters
(15.2 cm)

Ordering Information	- TAD 25
Part No.	Description
AW-1800	TAD 25 Hose Dryer with Curtain, Trays and Brackets, 115V
AW-1802	TAD 25/2 Hose Dryer only, 115V, 60 Hz
AW-1803	Removeable Air Flow Curtain Assembly
AW-1835	Air Flow Curtain
AW-1805	Bracket Strip-Assembly (per section)
AW-1875	Holding Tray Assembly (10" x 10")
AW-1844-1	6" diameter Filter, (dozen)
AW-1844-2	6" diameter Filter, (case of 100)