

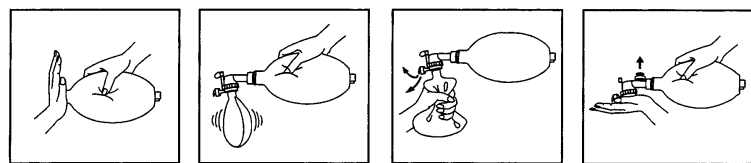
1. Compress the resuscitator bag with one hand, then occlude patient valve outlet with your other hand. Release the grip on the bag. Rapid bag re-expansion confirms efficient air intake.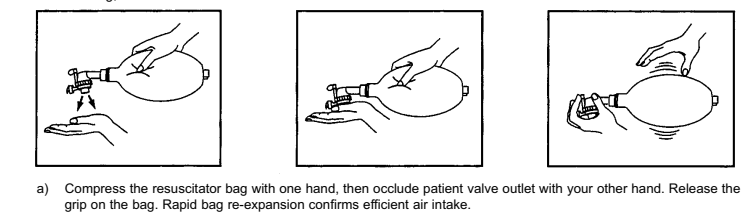

- Remove patient valve, close the neck opening and try to compress the bag. If the bag cannot be compressed with reasonable force, or if bag compression forces the air out between your hand and neck of the bag, the intake valve is efficiently preventing backward escape of air.

- INSPIRATION PRESSURE-LIMITING DEVICE. The infant and child resuscitators may feature a patient valve with a special pressure-limiting device mounted on the upper valve housing. If inspiration meets with pulmonary resistance, venting will occur, limiting pressure to 40 ±5-10cm H2O.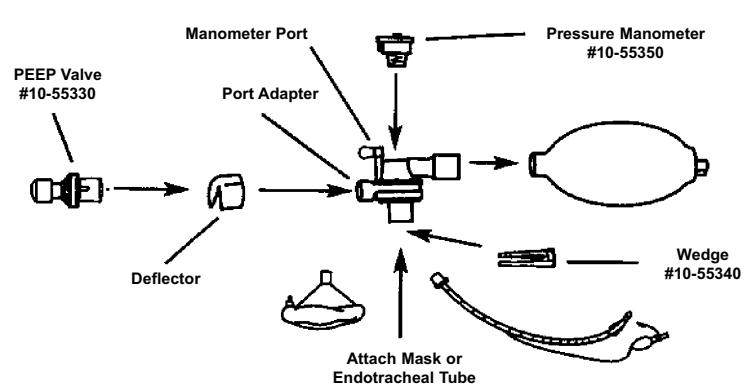



- Clear mouth of foreign matter. 2. Position patient for open airway. 3. Apply mask firmly to face to achieve a light seal.

OPERATING INSTRUCTIONS: Note: If these do not occur, patient's airway or patient valve may be blocked. Take immediate action by replacing the resuscitator or use an alternative procedure appropriate for situation.

Table with 3 columns: Specification Name, Value 1, Value 2. Rows include Adult, Child, Infant for various parameters like Volume, Stroke Volume, etc.

EXTERNAL RESUSCITATOR DIMENSIONS: (patient valve and bag only) MEASURED VOLUME: Bag Reservoir Adult: 34 cm long x 14 cm diameter

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RESPIRATORY RESISTANCE: 2.1 cm H2O @ 50 LPM Child. IMPACT TEST: The Mercury CPR-2 Bag showed no signs of damage, or degradation of performance as a result of a one meter drop onto a concrete surface.

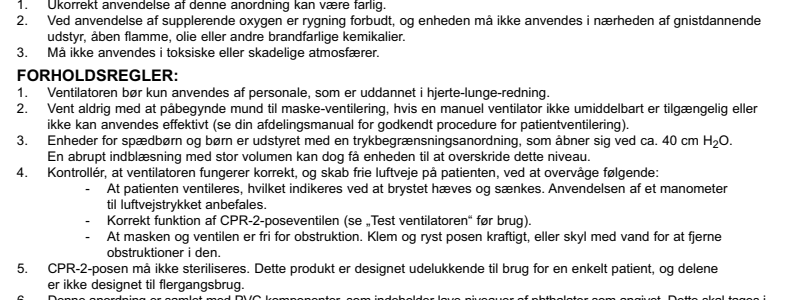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